



ABS14


SHOWER SEAT INSTALLATION INSTRUCTIONS

Before installing, please read the instructions carefully in order to install and use this product safely.




ABS14

PARTS LIST:

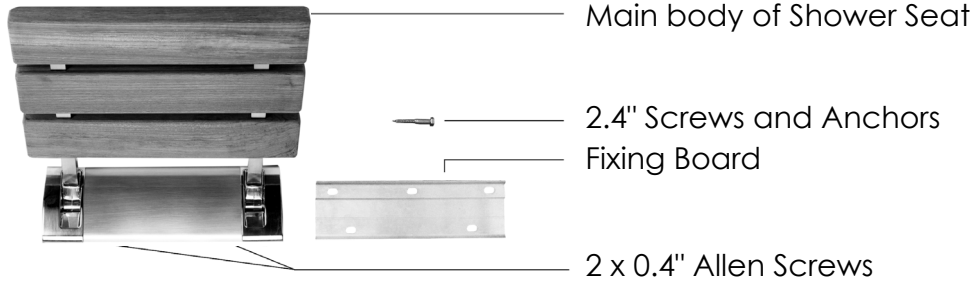


STEP BY STEP INSTALLATION:

- A. Place flat side of Fixing Board flush and leveled with the wall (Note: The edge with 3 holes should be up). The recommended height from the bottom of the Fixing Board to the floor is 15 1/2" - 19 1/2 ". Mark the location of the screw holes on the wall and remove the Fixing Board so as to drill 5 holes with a depth of about 2 1/2". Insert anchors into holes. (Figure A)
- B. Screw the Fixing Board into the wall and tighten. (Figure B)
- C. Align the main body of the Shower Seat with the Fixing Board on the wall. (Figure C)
- D. Use the supplied Allen key to tighten the 2 Allen screws already attached to the underside of the main body of the Shower Seat. (Figure D)

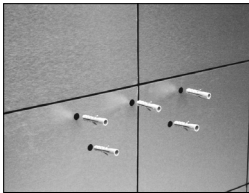


A

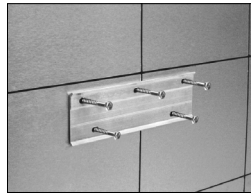

B



C

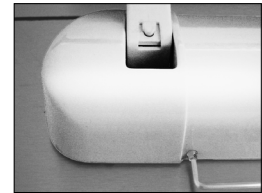

D



IMPORTANT:

When properly installed, the maximum safe loading weight on this seat is 300 lbs.